Elacin

SleepFit | Calm | Relax
Swim

EN | User Manual

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SE | Bruksanvisning

NO | Brukerveiledning

Custom-made earmoulds

Op maat gemaakte oordoppen

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Moldes hechos a la medida

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Brugertilpassede ørepropper

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
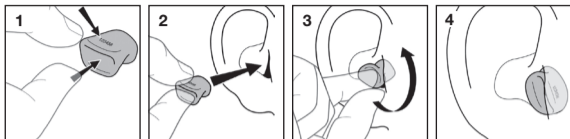
Spesiallagde ørepropper

1. **EN** | User Manual
2. **NL** | Gebruiksaanwijzing
3. **DE** | Gebrauchsanleitung
4. **FR** | Mode d'emploi
5. **ES** | Manual del usuario
6. **IT** | Manuale dell'utente
7. **DK** | Brugervejledning
8. **FI** | Käyttöohje
9. **SE** | Bruksanvisning
10. **NO** | Brukerveiledning

Elacin SleepFit | Calm | Relax

Elacin Swim

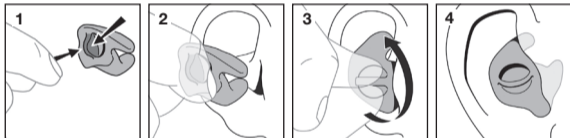Elacin Custom-made earmoulds

IMPORTANT: Read these instruction carefully before use and keep both the packaging and the user manual.

General

Elacin custom-made earmoulds are subject to the highest quality standard in its industry. Each product is unique and manufactured with maximum care to guarantee safety and comfort. Each pair of earmoulds is issued with an identification number.

Use

Check that the earmoulds are clean before you use them. The earmould with the letter 'L' is for the left ear. The earmould with the letter 'R' is for the right ear. Your hearing specialist will be glad to demonstrate the correct way to insert and remove the earmoulds.

Insertion

Insertion may be less smooth at first use. Application of included ear cream on the part of the earmoulds that goes into the ear will improve ease of insertion (see illustrations).

Removal

Remove the earmould using a rotary forward motion.
OR, alternatively,
Place your index finger against the lower part of your ear and press upwards a number of times. The earmould is then slowly released from the ear. When it no longer seals you can safely remove it. Immediately after use, put the earmoulds in the case supplied with them.

Maintenance

Clean your Elacin earmoulds regularly with a damp cloth. For a regular and thorough cleaning of your

custom-made earmoulds, we advise you to use the Elacin Hygiene and Hygiene^{PLUS} kit. These cleaning products help you to keep your earmoulds in the best possible shape and prolong the life-span. For more information ask your hearing specialist.

Warning

Use Elacin earmoulds carefully, in accordance with the instructions.

The use of chemicals may damage your Elacin earmoulds. Ask your supplier for more information.

Sudden or fast removal of the earmoulds out of the ear canal may damage the ear drum.

Changes in body weight can affect the fit of an earmould to the extent that it will not fully close off the hearing canal.

The earmoulds are custom-made and are therefore not transferable.

The earmoulds are designed to be inserted into the hearing canal only.

Replace your earmoulds after four years.

The greatest possible care is taken in the manufacturing of your Elacin earmoulds.

Contact your supplier immediately if you notice anything out of the ordinary.

Elacin SleepFit | Calm | Relax

1. Hold the earmould between your thumb and index finger, with your index finger in the hollow and your thumb on the identification number.
2. Turn the earmould such that your thumb is pointing downwards and hold it by your ear.
3. Insert the earmould by rotating it backwards and, using gentle pressure, putting it into the ear canal. Your thumb is now pointing forwards.
4. Using gentle pressure from your index finger, move the earmould up and down slightly until it seals comfortably.

Elacin Swim

1. Hold the handle of the Elacin Swim earmould between your index finger and thumb. The index finger is on the top with the thumb underneath.
- 2.+3. Insert the point of the Swim earmould in your ear and, using gentle pressure, rotate it backwards and put it into the ear canal.
4. The handle is now horizontal in your ear.

Warning: The Swim earmoulds should not be used for diving.

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Elacin is a registered trademark of Elacin International B.V.
Elacin International B.V.